## SpermRinse™

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### EN

Medium for sperm preparation.

**Product Description**
SpermRinse™ is a bicarbonate and HEPES buffered medium containing human serum albumin and gentamicin as an antibacterial agent.

For use after equilibration at +37°C and 5 % CO<sub>2</sub> or ambient atmosphere.

**Storage instructions and stability**
Store dark at +2 to +8 °C.

SpermRinse™ is stable until the expiry date shown on the container labels and the LOT-specific Certificate of Analysis.

Media bottles should not be stored after opening. Discard excess media after completion of the procedure.

**Directions for use**

**Swim up preparation**
Using a sterile, non-toxic pipette, add 1.0 mL of semen to a tube labelled with patient ID.

Overlay 2.0 mL of SpermRinse™. Place the tube in incubator at +37°C and 5 % CO<sub>2</sub> for 30-60 minutes or heat the tube to +37°C for 30-60 minutes. Aspirate the top medium and transfer to a clean tube. Add 5.0 mL of SpermRinse™, mix and centrifuge for 10 minutes at 150–1000 g.

Aspirate the supernatant and re-suspend the pellet in 5.0 mL of equilibrated IVF™ to remove the gentamicin and equal the centrifugation. Discard the supernatant and suspend the sperm in 1.0 mL of equilibrated IVF™.

**Gradient separation**

Mix SpermRinse™ with SpermGrad™ to obtain 90 % and 45 % gradient concentrations, respectively. Using a sterile, non-toxic pipette, place 1.5 mL of 90 % solution into a conical centrifuge tube first and then slowly add 1.5 mL of 45 % solution on top. Finally, the semen is gently layered on top of the gradient. Centrifuge the tube for 20 minutes at 150–1000 g. Remove the semen residue and two top layers, taking care not to leave residue on the tube wall. Transfer the sperm pellet with as little of the 90 % solution as possible to a sterile conical tube containing 5.0 mL of SpermRinse™. Centrifuge for 10 minutes at 150–1000 g. Discard the supernatant and wash one more time with SpermRinse™. After the second wash, re-suspend the pellet in 1.0 mL of equilibrated IVF™.

**Specifications**
Sterile filtered SAL 10<sup>+</sup>
Mouse Embryo Assay (1-cell) [% expanded blastocyst within 96 hours] ≥ 80
Bacterial endotoxins (LAL assay) [EU/mL] < 0.25

LOT specific test results are available on the Certificate of Analysis provided with each delivery.

### Precautions

Discard product if bottle integrity is compromised. Do not use SpermRinse™ if it appears cloudy.

SpermRinse™ contains human serum albumin and gentamicin.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived was found negative when tested for antibodies to HIV, HBC, HCV, and HTLV I/II and non-reactive for HbsAg, HCV RNA and HIV-1 RNA and syphilis. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

Re-use may result in microbiological contamination and/or property changes in the product. To avoid contamination Vitrolife strongly recommends that media should be opened and used only with aseptic technique.

The risks of reproductive toxicity and developmental toxicity for IVF media, including Vitrolife’s IVF media, have not been determined and are uncertain.

Caution: Federal (US) law restricts this device to sale by or on the order of a physician.

STERILE A	Sterilized using aseptic processing techniques
~4~	Temperature limit
⊘	Do not re-use, discard after procedure.
⊞	Use by - see label.
⚠	Caution: Consult accompanying documents.
REF	Catalog number
LOT	Batch code
CE 2460	CE Mark (Conformité Européen)

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